



BUCCAL DNA COLLECTION KIT (BDK)

©ITSIBIO2012: Validated BioLab Procedures

IMPORTANT: K-0022 is a customized kit validated specifically for collection of buccal DNA sample. The ITSI-Bioscience buccal DNA collection kit allows for the simple and accurate collection of cheek cell samples from which DNA will be isolated. The kit provides components for collecting, documenting and shipping a DNA sample. Each kit contains detailed instructions, cotton-tipped applicator, gloves tamper evident tape and cardboard mailer for shipping.

Read the instructions completely before starting.

MATERIALS PROVIDED IN THIS KIT:

Item	Number
Sterile Cotton-Tipped Applicator	2 x 2
Gloves	1 Pair
Tamper Evident Tape	1
Instructions	1
Cardboard mailer	1

INSTRUCTIONS FOR COLLECTING A BUCCAL DNA SAMPLE

- Put on the enclosed latex gloves.
- Peel open one end of the wrapper of the enclosed sterile swab packages and remove the two cotton-tipped swabs. **OPEN THE END INDICATED ON THE WRAPPER. SAVE THE WRAPPER**
- With gentle pressure, roll one of the cotton-tipped swabs against the inside of one cheek. Repeat with the other swab.
- Repeat step #2 and #3 for the second swab package on the opposite cheek.
- Allow the swabs to air-dry for at least five minutes. Do not let the cotton tip touch any surface during drying.
- Return the swabs to the original wrappers and place them in the white cardboard mailer. Discard the gloves.
- Seal the flap of the mailer with the tamper-evident tape provided. With a pen, place your initials and the date across the tape.
- Return the Reference Buccal DNA Collection Kit to ITSI-Biosciences Lab, either in person or by any courier service that uses a tracking number. Use the enclosed

blank label if the kit will be sent to a different laboratory for analysis.

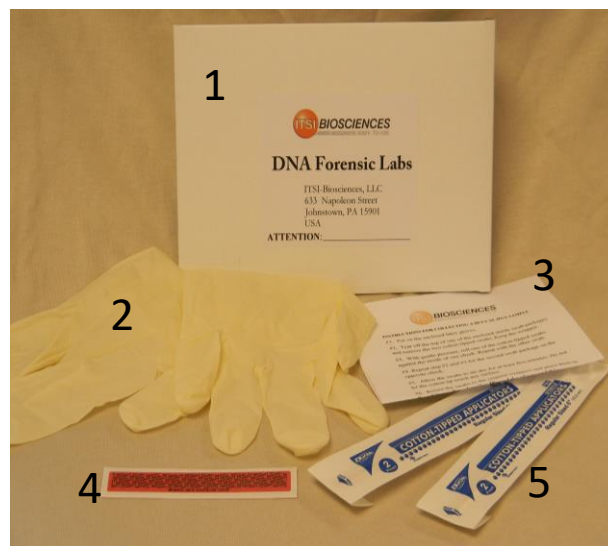


Figure 1: Buccal DNA Collection kit, showing Cardboard Mailer (1), latex gloves (2), protocol (3), tamper-evident seal (4) and cotton tipped applicator used for cheek cell collection (5). Note: the blank label is not shown in this picture.

Distributed Exclusively By:

ITSI-Biosciences, LLC
DNA Analysis Laboratory
 633 Napoleon Street
 Johnstown, PA 15901 USA.
 Attention: DNA Analysis Manager
 Phone: 814-262-7331 Fax: 814-262-7334
 Website: www.itsibio.com Email: itsi@itsibio.com